


















Acorn / Brooks Model 180

Digital Display Codes

The stairlift is switched off or in sleep mode, <i>see page 20.</i>	
The stairlift is ready to use but the batteries are not being charged - Check mains electricity supply is switched on and the stairlift is parked on its charge points.	
Moving Up (arm switch control)	
Moving Down (arm switch control)	
Moving Up (Remote control)	
Moving Down (Remote control)	
Lift waiting for hinge to operate	
Batteries are charging	
Batteries are fully charged	
Down side foot sensitive safety edge has been activated - Check for obstructions.	
Up side foot sensitive safety edge has been activated - Check for obstructions.	
Under foot plate sensitive safety edge has been activated - Check for obstructions.	
Down side sensitive safety edge has been activated - Check for obstructions.	
Up side sensitive safety edge has been activated - Check for obstructions.	
The seat is swivelled away from its operating position.	
Battery low – Bring the unit down to the nearest charging station. The unit won't allow the user to travel in the UP direction until the batteries are sufficiently charged and the code has cleared.	
Battery flat – Audible alarm – Press the arm switch control or the remote control once to stop the alarm. Bring the unit down to the nearest charging station.	
If this symbol is displayed, DO NOT use the hand winder in the down direction as this could damage the stairlift. Contact your stairlift supplier immediately.	